Dubliplast®

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EN Warranty

Dentaurum guarantees a faultless quality of the products manufactured by us. The information in these instructions for use is based upon our own experiences. The user himself is responsible for the correct processing or use of the products. We cannot be held responsible for failures, as we have no influence on the processing. However, any claim for compensation only applies to the commercial value of our products.

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Dear Customer

Thank you for having chosen a quality product from Dentaaurum.

In order to use this product at its best for you and your patients, it is important to study and follow these directions for use carefully.

The written instructions cannot cover all eventualities during operation. For this reason our Hotline is available to answer any other questions and ideas that may arise.

Our products are regularly modernized to the latest technical development. For this reason we advise examining the most up-to-date operating instructions supplied for any possible changes, even if the product is frequently used.

Instruction for use Dubliplast®

Liquid Concentrate Duplicating Material

Dubliplast® is an elastic duplicating material for use with plaster, eg. Rapidur®, plaster containing investment material and phosphate bonded investment material, eg. rema® Exakt or rema® dynamic (top speed).

Dubliplast® is a reversible hydrocolloid manufactured from high quality, natural ingredients and stabilizing substances. Preservatives protect Dubliplast® against fungal attack and guarantee long life when handled correctly.

Dubliplast® is used for the production of duplicate models made of plaster, plaster containing investment material or phosphate bonded investment material. Dubliplast® mold surfaces do not need to be treated with any special solution. Dubliplast® is very fluid, volume stable and the contours of the duplicate model are sharply defined. The surface of the duplicate model is smooth. Dubliplast® is also suitable for producing refractory models for framework castings.

Initial melting

1. Cut Dubliplast® into small pieces and put it into the duplicating unit without adding water.
2. Set the melt temperature on automatic duplicator to 93 °C / 199 °F and the pouring temperature to 50 °C / 122 °F.
3. If the double boiler method of preparation is used, the mixture must be stirred constantly. Do not bring the mixture to a boil.

Melting temperature: 93 °C / 199 °F

Pouring temperature: 50 °C / 122 °F

Duplicating – general notes

Remelting after first use

Clean Dubliplast® carefully under running water. Cut into small pieces and place into automatic duplicating machine or double boiler. Do not add water to gel.

Model preparation

Prepare model in normal fashion using heat resistant preparation and/or block-out waxes such as Dentaureum's preparation wax, REF 120-025-00, and/or Thermo-Wax®, REF 120-170-00.

Soak model prior to duplication

Clean model and place in warm (40 – 45 °C / 104 – 113 °F) water, teeth down, for 30 minutes.

Duplicating

After soaking remove excess water from model using a clean tissue or paper towel. Do not use compressed air. Secure model to metal duplicating flask base and put top into place. Use Dentaureum's metal flask, REF 165-700-00.

Duplicating material should be stirred continually during melting and cooling.

Pouring temperature to 50 °C / 122 °F (± 1°C / 2 °F) .Carefully pour duplicating material into one side only of the metal flask.

Allow flask to bench cool for 15 minutes before placing in water bath. Water level should be approx. 2 cm and the temperature between 18 – 25 °C/64 – 77 °F.

Allow flask to remain in water bath for approx. 30 minutes.

Note:

If a plastic duplicating flask has been used the setting time should be extended for up to 30% of regular time.

After setting, the master model is carefully removed using model tongs, REF 008-030-00. The model should be lifted straight up out of the mold to avoid damaging the duplicating gel. Investment material is then mixed according to the manufacturer suggested ratio and after max. 1 hour poured at room temperature (20 °C – 24 °C / 68 °F – 75 °F) into the clean, dry mold. After the required setting time of 40 minutes, the duplicating material is removed from the flask and then carefully peeled away from the model.

Remelting Dubliplast®

Dubliplast® may be melted repeatedly. Correct use and storage will prolong its durability. Clean the used Dubliplast® material under running water immediately after the models have been removed and keep it in a clean and airtight container. Cut Dubli-

plast® into small pieces and put into the automatic duplicating unit without adding water. To prevent loss of moisture, keep the lid of the automatic duplicating unit closed.

Note:

Dubiplast which has been used for phosphate casting investments must not be used for plaster or plaster-containing investment materials.

Note:

Never mix new Dubiplast® with previously melted material.

Sources of faults and their remedies

Fault: Soft, scaly model surfaces.

Cause: Duplicating mold too moist.

Remedy: Do not duplicate models which are too moist.

Cause: Setting times of duplicating material too long or too short.

Remedy: Observe instructions for use for the duplicating materials.

Cause: Original models were contaminated (alginate + remainders of saliva).

Remedy: Models must be steam-cleaned before duplicating.

Fault: Poor ultimate tensile strength.

Cause: Duplicating material has been contaminated by alum, electrolyte or washing materials.

Remedy: Material is permanently damaged and should be discarded.

Fault: Smearly, soft consistency of Dubiplast®.

Cause: Old duplicating material.

Material boiled away.

Contamination.

Foreign substances, e.g. cleansing agents.

Remedy: Material is permanently damaged and should be discarded.

Fault: Viscous duplicating material.

Cause: Proportion of water is evaporated.

Remedy: Add 10–30 ml of tapwater (per kg duplicating material).

Recommended melting units

duplicating unit (Dublitherm), microwave unit

Change of duplicating materials

Never mix old and new materials. When changing materials, clean the melting unit completely and fill it with new material. Do not use cleansing solvents.

Shelf life of Dubliplast®

60 months in the unopened original packaging stored at room temperature.

Waste disposal

The duplicating gel may be disposed together with normal domestic refuse.
The empty packaging should be disposed after use according to local regulations.

Availability

Dubliplast®	1 kg	REF 165-100-00
Dubliplast®	5 kg	REF 165-500-00

Accessories for duplicating

Duplicating flask Non-oxidizing metal flask for short gelling times with holder for sprueformer.	REF 165-700-00
Model lifting tongs	REF 008-030-00

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